


HOW TO

Install the eRAD Evo Image Viewer

1

Log in to Connect and navigate to the Account section.

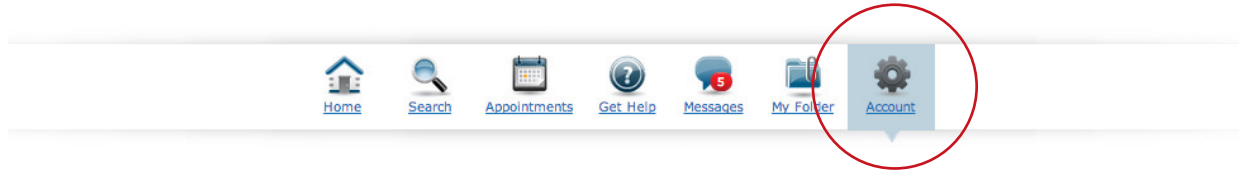

2

Navigate to the Image Viewer tab and click the Download Full Viewer button.



DOWNLOAD FULL VIEWER

3

Go to your downloads folder and launch the [EVO7Viewer.exe](#) and follow the installation instructions.

After the installation you will be able to launch images.

If for any reason images do not launch on first try, please reboot the workstation. You will see the viewer icon in the system tray near the time/date section on the bottom right. You can now launch studies using the Evo viewer.

